

MAGLUMI™ 2000

Chemiluminescence Immunoassay (CLIA) System

Technical Specifications

Reagent Feature

- Flash chemiluminescence label –ABEI, with high sensitivity and long stability
- Magnetic microbeads separation, fast and efficient

Main Features

- Throughput: up to 180 tests/hour
- 24 hours ready-to-use
- Time to first result: 17 minutes

Modes of Operation

- Random, Batch and STAT

Sample Loading

- Up to 144 samples tubes
- Continuous loading, STAT available
- Barcode reader recognition or analyzer auto numbered
- LIS connection, auto read sample info

Reagent Loading

- 15 reagents on board
- Continuous loading
- RFID reading all info of reagent
- Refrigerated reagent area with independent power supply

Reagent Integral

- Ready-to-use, no pretreatment required
- Including calibrators
- RFID tag storing all info of reagent
- RFID tag with built-in master curve
- 2-point calibration to adjust master curve
- Calibration stability: Max 4 weeks

Other Features

- Clot detection
- Liquid level detection
- Auto and optional ratio for high sample dilution
- Constant 37°C incubation

Operation System

- Windows 7
- HP compatible (Dual-core CPU)
- Color touch screen

Interconnection

- Bi-communication with LIS via ASTM protocol

Dimensions Weight

- Input: AC 110-230V, 50/60HZ
- Dimension and weight: 135 X 76 X158cm 158kg



Shenzhen New Industries Biomedical Engineering Co., Ltd
(SNIBE Co., Ltd)

21st Floor, Block A, Building 1, Shenzhen Software Industry Base,
No. 1008, Keyuan Road, Nanshan District, Shenzhen 518000 CHINA

Tel: +86 755 26501514 Fax: +86 755 26654850

Email: sales@snibe.com Web: www.snibe.com



DISTRIBUTOR:

M1005E01-171010



MAGLUMI™ 2000

Chemiluminescence Immunoassay (CLIA) System

FDA
510(k)
Cleared



www.snibe.com

Outstanding Technology Power of MAGLUMI™

Chemiluminescence Immunoassay (CLIA) System

CLIA uses two important technologies, one is labeling technology which determines reaction mode; and the other is separation technology which determines the sensitivity, accuracy and precision of the reagents.

Labeling Technology

Two types of labeling technologies are commonly used. One is enzyme label and the other is non-enzyme small molecule label. Enzyme label reagents are not so stable and are easily affected by the change of storage conditions. Maglumi system applies ABEI labels. ABEI is a non-enzyme small molecule with special molecular formula to enhance stability in acid and alkaline solution. As fast chemiluminescence, ABEI chemical reaction with sodium hydroxide (NaOH) and Hydrogen peroxide (H₂O₂) finishes the process in 3 seconds.

Separation Technology

Maglumi uses Magnetic Microbeads. As separation technology, it has been widely used in the field of CLIA. Compared with traditional separation technology, it has the following advantages:

- Shortening the reaction time by enlarging the reaction area of antigens and antibodies.
- Enhancing the sensitivity by better and faster capture of antigens and antibodies.
- Reducing inter or intra-assay discrepancies significantly by mixing the reagents thoroughly in a liquid separation platform.
- Enhancing the accuracy by absorbing antigens and antibodies through chemical reaction.



MAGLUMI™ 2000



Pipettor

- Aspirate reagent/sample with high speed
- Titanium needle for crushproof
- Clot detection & liquid level detection
- Coated with TEFLON to prevent carry-over
- Inner & external washing




Automated cuvette loader

- Capable of loading modules continuously
- Up to 180 reaction modules



Sample area

- Up to 144 sample on board with barcode
 - Continuous loading & cooling function
- Random, batch, and STAT modes

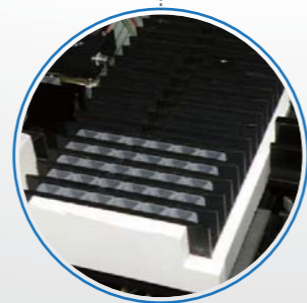

Incubator

- 16 Incubate slots (96 tests at the same time)
- The temperature is $36.8\text{ }^{\circ}\text{C} \pm 0.5\text{ }^{\circ}\text{C}$




Chamber

- High-sensitivity and low-noise PMT
- Cuvettes detection and anti-overflow



Washer

- Strong magnetic field helps better separation
- Three pairs of washing needles with anti-overflow function

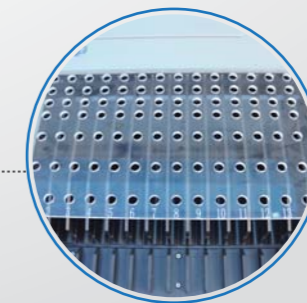

Reagent area

- Cooling function
- 15 Reagents on board, continuous loading
- RFID label storing reagent information
- RFID tag, high reliability
- Easy and fast for adding and updating

Operation Software



Comprehensive software

- User-friendly Interface
- Real-time status monitoring for each test
- Monitoring reagent and consumable status
- Intelligent alarm function



Maintenance Guide

- Daily, weekly and monthly maintenance guide for analytical unit
- Ensure performance reliability



Quality control

- Westgard rules and Levey-Jennings chart for both internal and external quality control



Test Summary Function

- Test summary including system test, calibration, QC, statistics of samples, valid tests and rerun tests
- Search and review test information conveniently



Pre or post defined dilution Function

- Pre or post defined dilution flexibly if high concentration is met



Connect to LIS (bidirection)

- Bidirection, Lis can connect by online and ASTM